


**Visage is launched from the Visage Icon or the "View Image" button in iCentra.**

## Mouse Controls




Scroll Slices

Click LEFT mouse button and drag

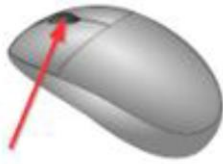


Scroll Images

Scroll mouse WHEEL

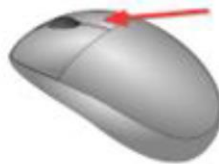


Pan Image

Click RIGHT mouse button and drag

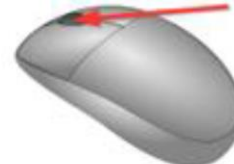

Zoom Image

Click WHEEL as a button and drag


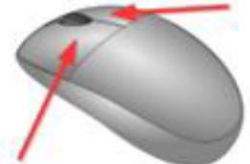


Window Level

Click both buttons and drag

To open the **Tool Palette**, right click on the image:

To configure the **Tool Palette**, select the **Configure Tool Palette** tool, click and drag tools to the desired location, and click **Ok**.

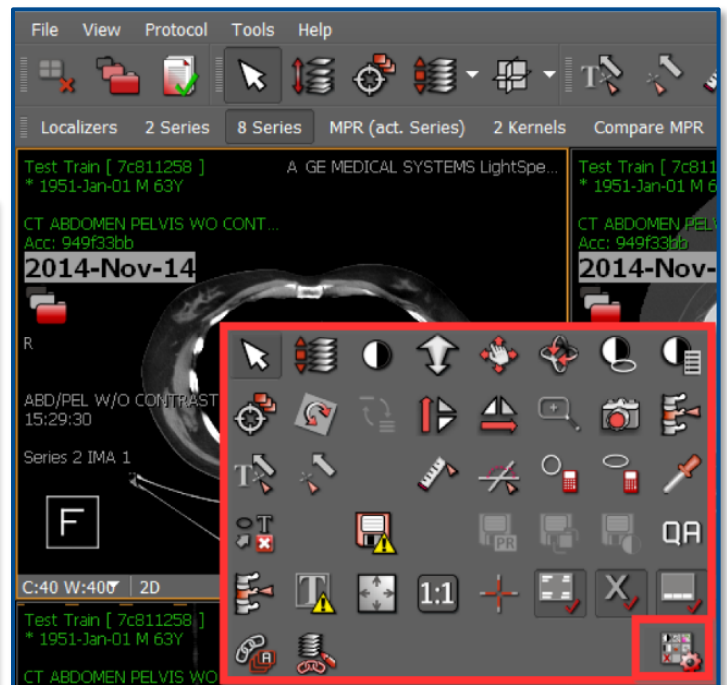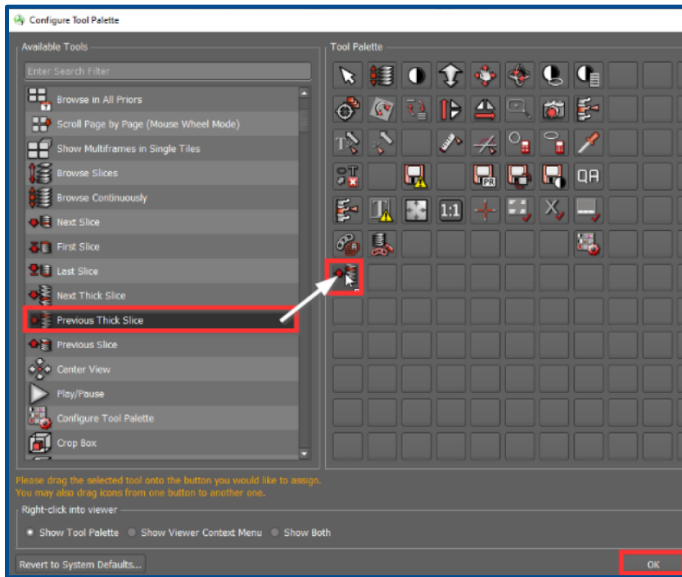

**NOTE:** The function of the tools can be found in the **Online Help Manual** by right clicking on the tool and clicking **Help**.

Training videos can be accessed through [Intermountain.net -> Imaging -> Visage](http://Intermountain.net -> Imaging -> Visage).

Scan the QR Code to be directed to the Visage **View Window** training video.

## Contacts

For Imaging support call (801) 442-6018 option 1

To learn more about this project, please contact [intermountainpacs@imail.org](mailto:intermountainpacs@imail.org).

