


Deleting Images from Olympus Processor


The Olympus Processor has a limited amount of internal memory. This process demonstrates how to delete all of the images from the Olympus Processor.

Delete Images from Olympus Processor


1. On the Olympus keyboard, hit the **Menu** button from the right side of keyboard

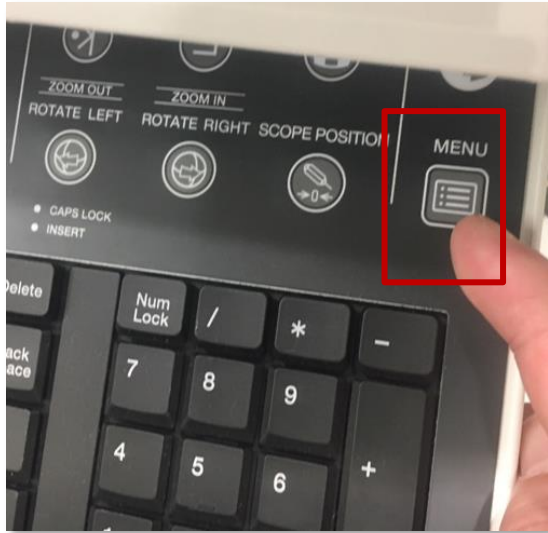


2. Use the arrow buttons to select **View Saved Images**. Hit enter.

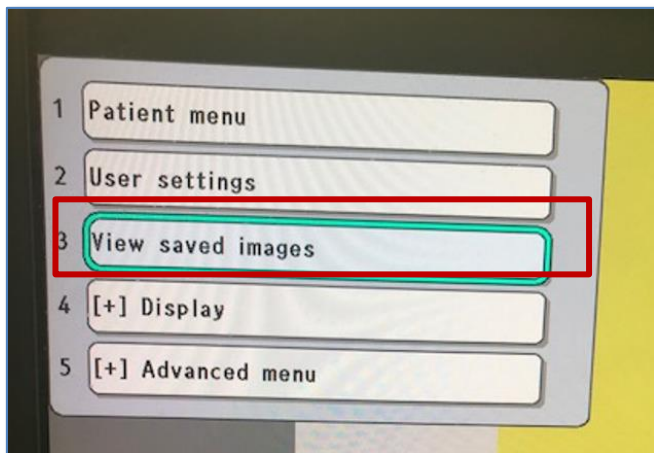

3. Use the arrow keys to navigate to **Manage Images**.

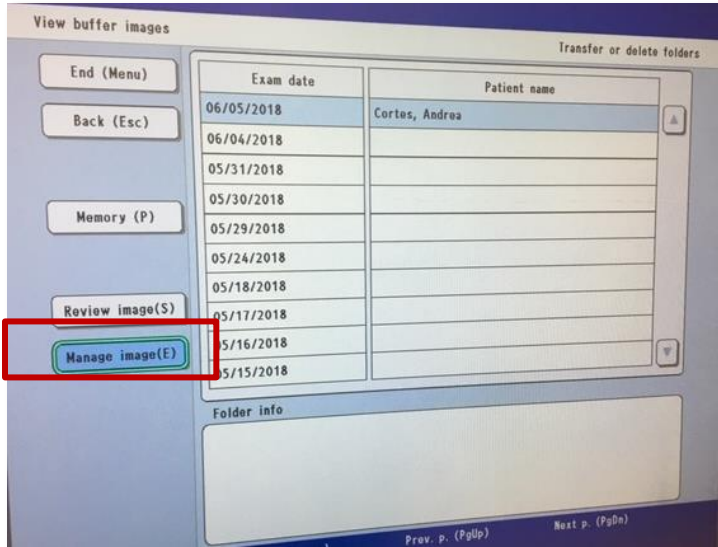

4. Use the arrow keys to navigate to **Delete** then over to **ALL**. Hit **R** on the keyboard.

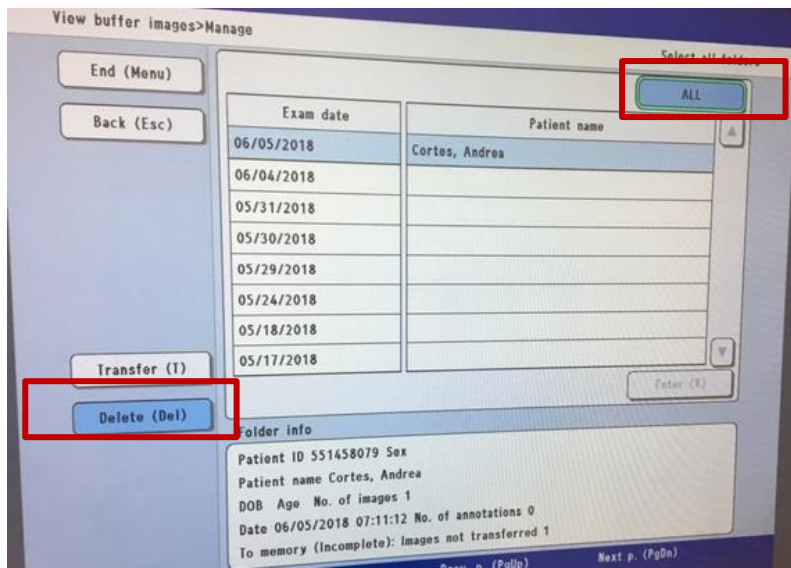

5. Select **Yes**. Hit **Enter**.

