



Manually Enter a Patient: Olympus CV-190 Processor


1. On the Olympus keyboard, hit the **Menu** button from the right side of keyboard.

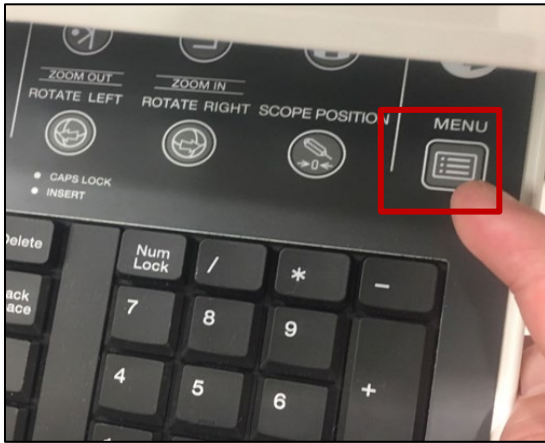

2. This image below will appear, select **PATIENT MENU**.

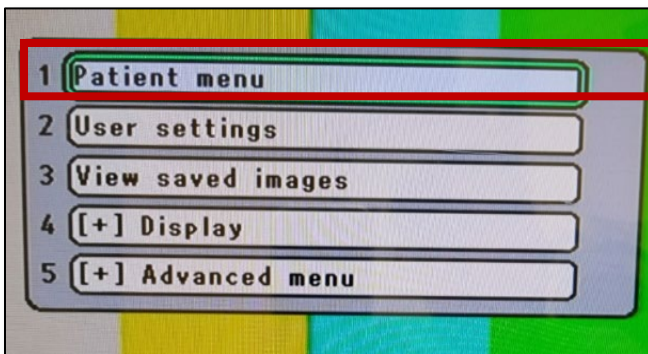

3. Use the arrow buttons to choose **EDIT**, hit enter. Now hit enter on 01.

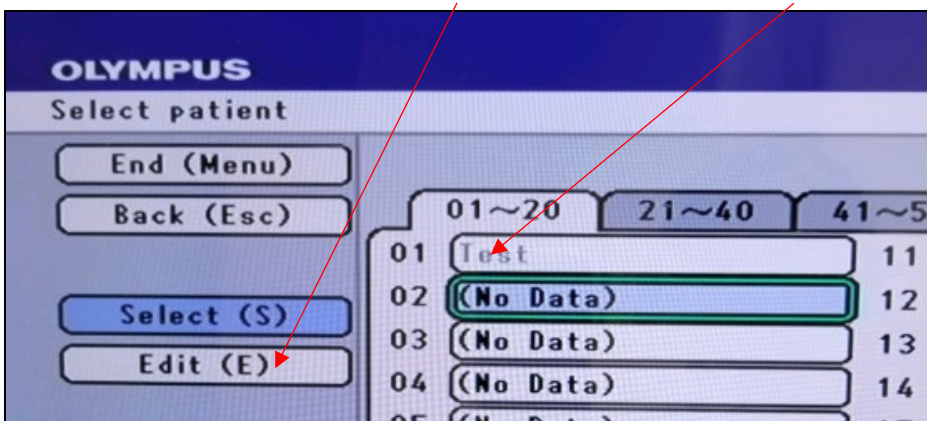

4. The following image will appear. Manually enter in the Patient Data.

OLYMPUS
Select patient>Edit>Input

02

Patient ID 0000000

Patient name xtest

DOB 01/01/2000

Sex m

Age 18

End (Menu) Back (Esc) Next

Move (↑ ↓ → ←) Select (Enter)

5. Once the patient data has been entered, push **ESCAPE** on the keyboard. The following image will appear and then select **YES**.

OLYMPUS
Select patient>Edit>Input Input new patient data

Do you save data and perform the following registration?

Yes No

Register next new data

End (Menu) Back (Esc) Next

Move (↑ ↓ → ←) Select (Enter)

6. You will now be taken back to the main menu. Select **PATIENT MENU** again.

1 Patient menu


2 User settings

3 View saved images

4 [+] Display

5 [+] Advanced menu

- Use the arrow buttons to choose **SELECT**, hit enter. Now select the patient.

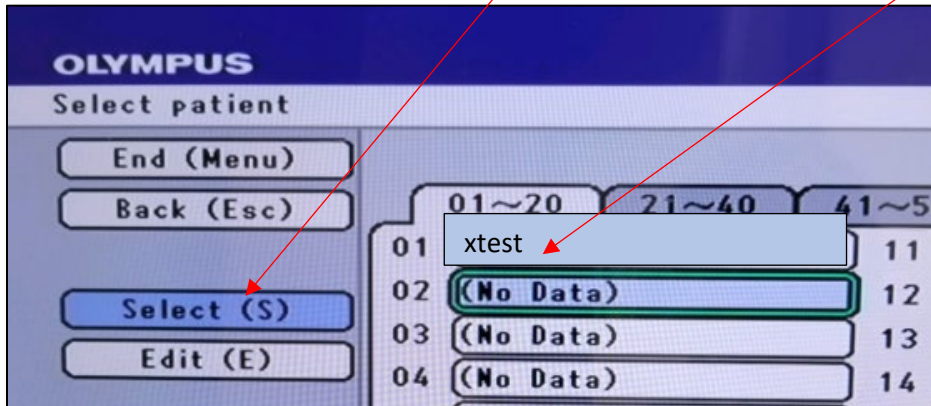

- You are now ready to capture images on the Olympus Processor.

