


Recovering Images from Olympus Processor


The Olympus Processor has a limited amount of internal memory. This process demonstrates how to retrieve the images.

Recover Images from Olympus Processor


1. Press **Patient Menu** on the Olympus (White) Keyboard.

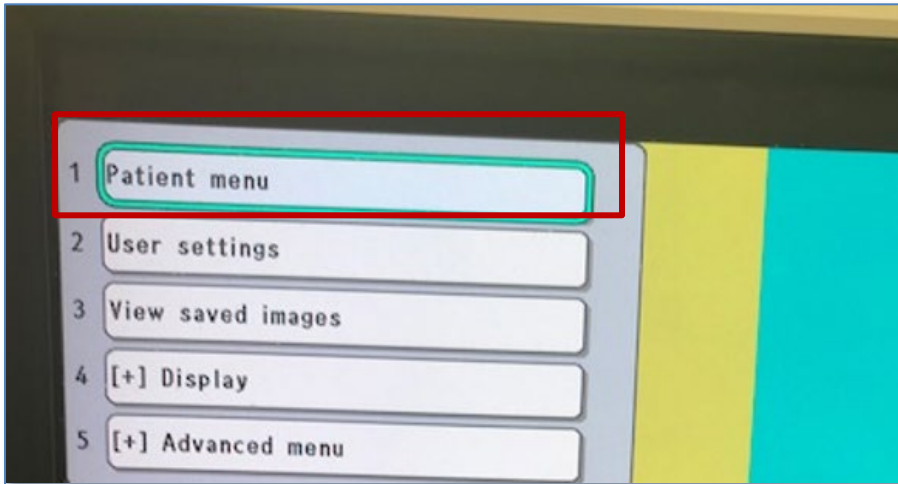


2. Scroll down to **#3 View Saved Images** and hit **ENTER** on the **white keyboard**.

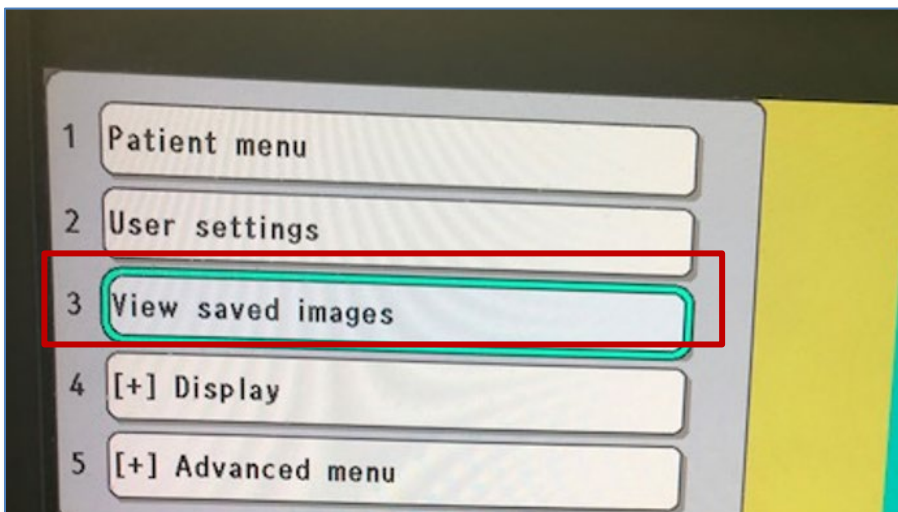


3. Arrow down to **Review Image** then arrow right to your patient. Hit **ENTER** on the **white keyboard**.

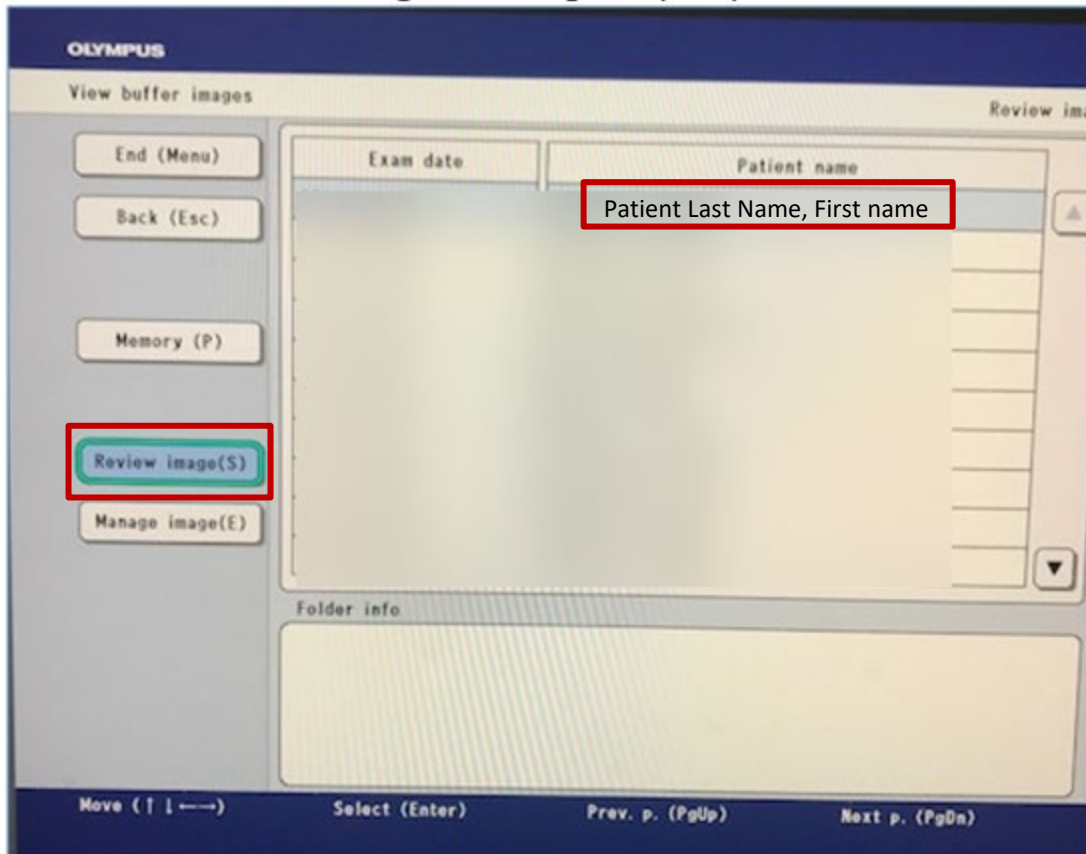


4. Arrow down to **View** then arrow right to **All**. Hit **ENTER** on the white keyboard, and then press the letter **"R"** on the **white keyboard**.

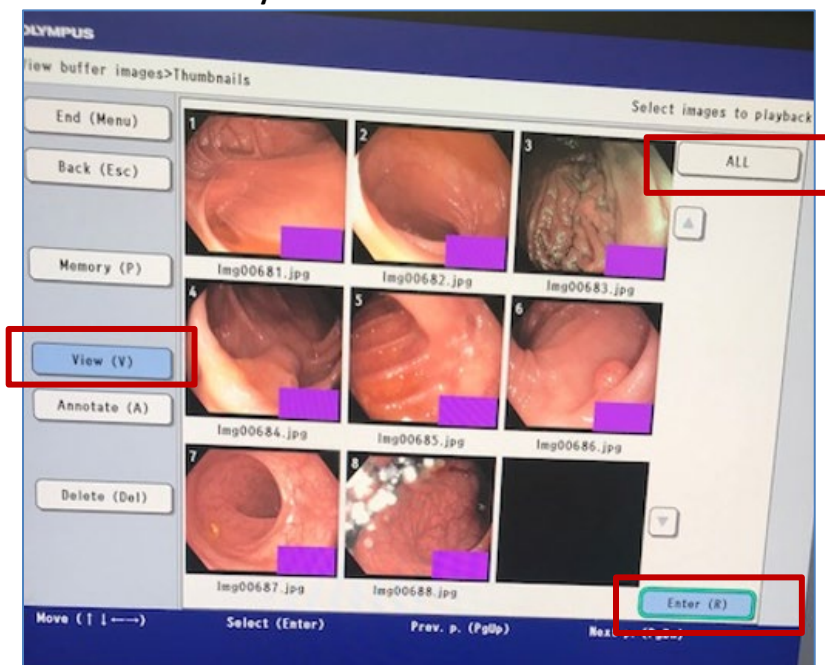


- The image will go full screen. Arrow right on the white keyboard to get to the next image.

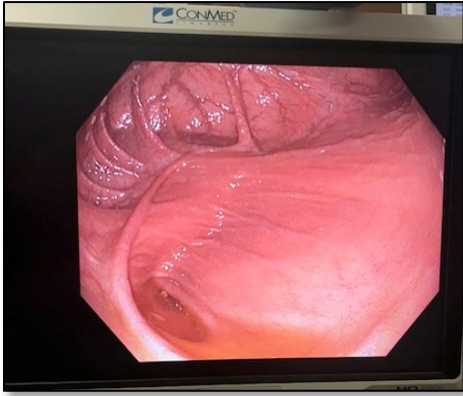


- Now go to the patient in ProVation.
- Select **GI** in ProVation and then **Procedure Documentation**.

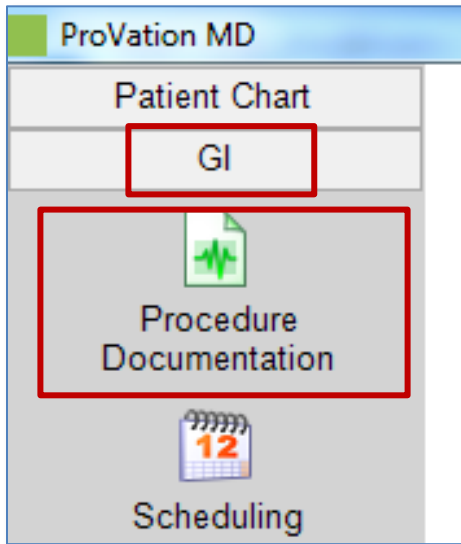

- Select your **Patient** and press the **Capture** button.

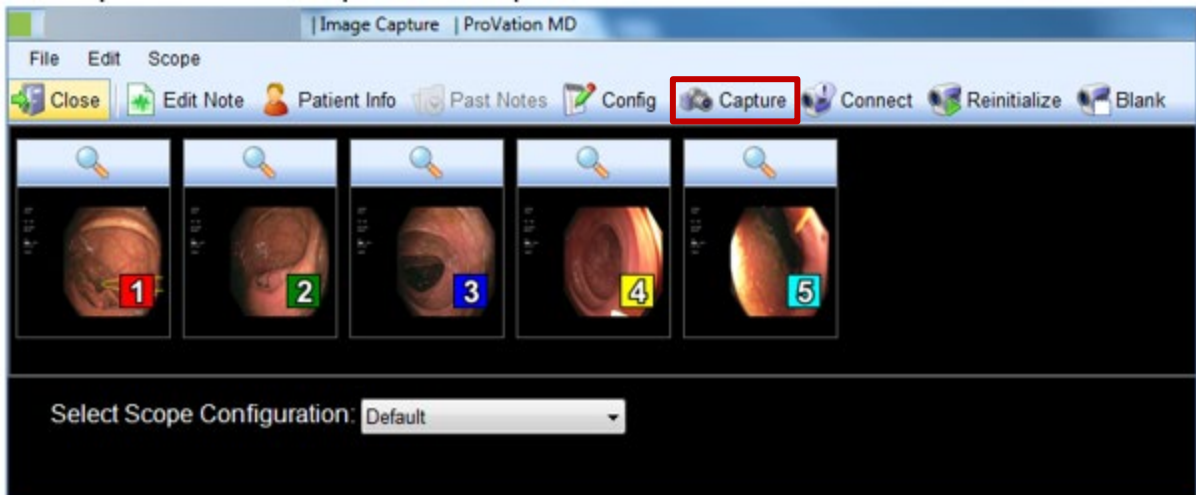

9. Once the above screen is up, press **"P"** on the **black keyboard** or select **CAPTURE** with the mouse.
10. After you have captured all the needed images, select **Close** in ProVation and go to your next patient.

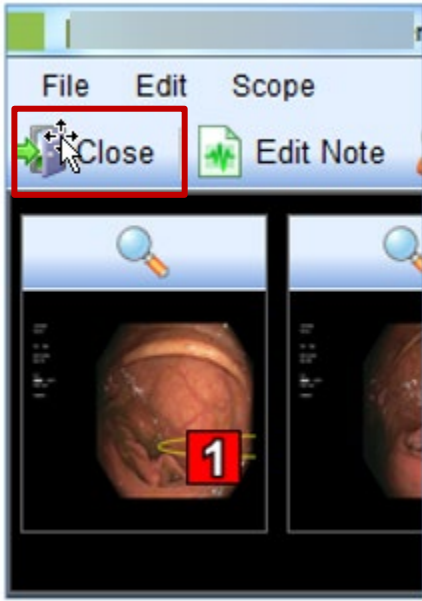

11. Now go back to the white keyboard and press **ESCAPE** all the way back to the main screen.