


Recovering Images from Olympus Processor


The Olympus Processor has a limited amount of internal memory. This process demonstrates how to retrieve the images.

Recover Images from Olympus Processor


1. Press **Patient Menu** on the Olympus (White keyboard) Keyboard.

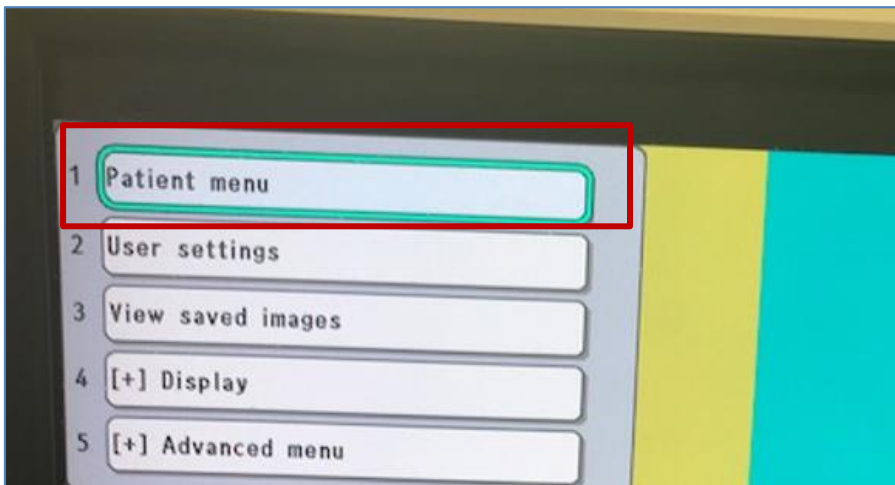


2. Scroll down to **View Saved Images** and hit ENTER on the **white keyboard**.

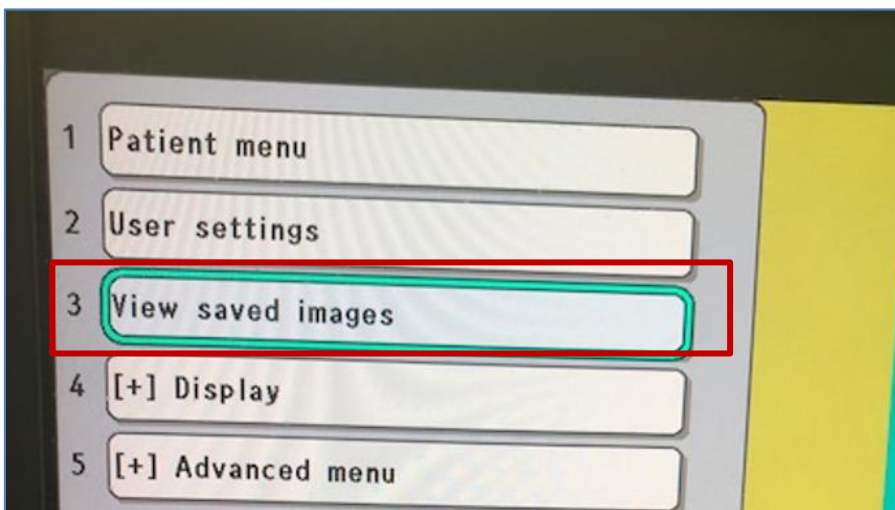


3. Arrow down to **Review Image**, arrow right to your patient, and hit **ENTER** on the **white keyboard**.

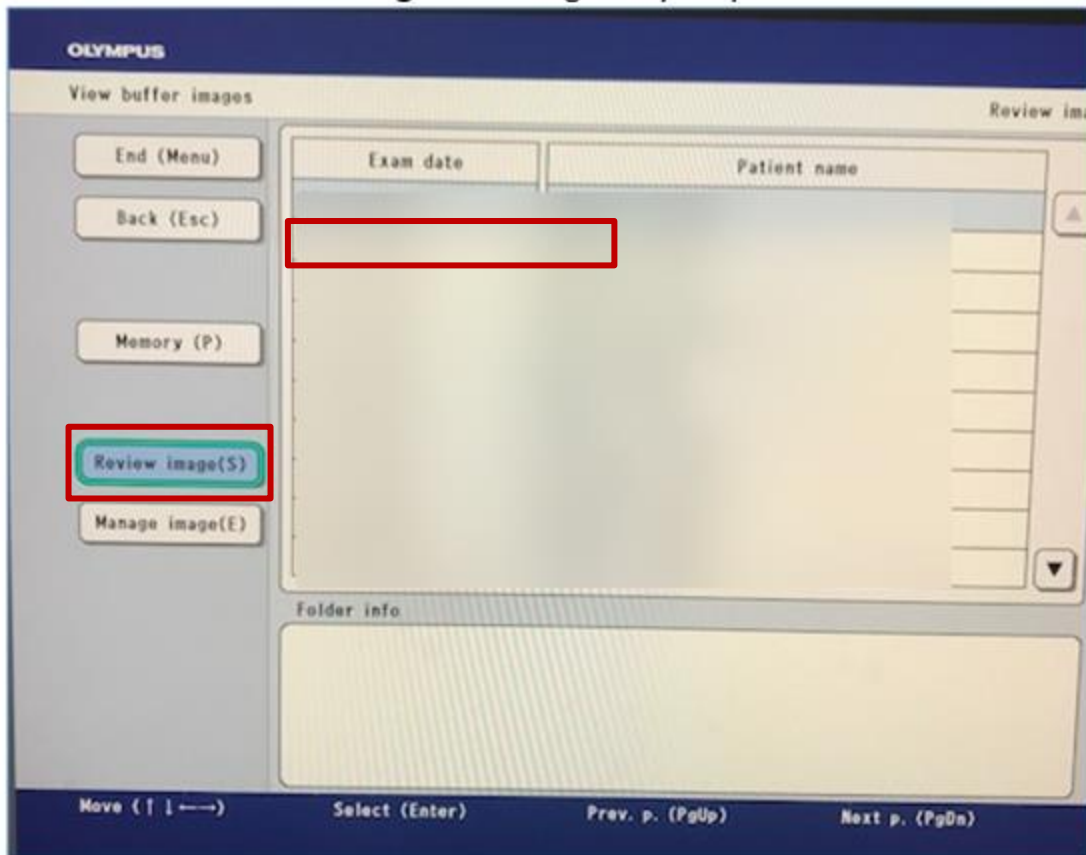


4. Arrow down to **View**, arrow right to **All**, hit **ENTER** on the **white keyboard**, and then press the letter **R** on the **white keyboard**.

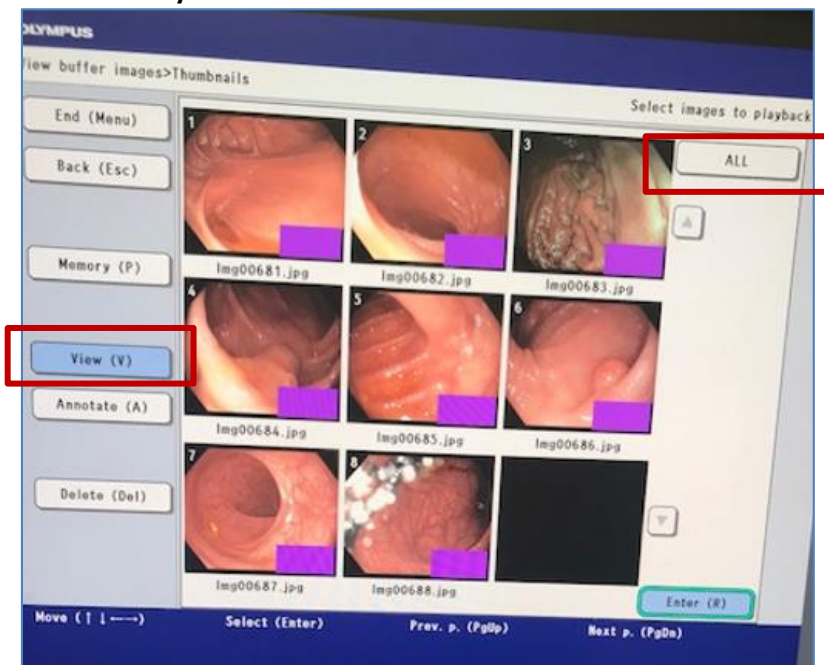


- The image will go full screen. Arrow right on the white keyboard to get to the next image.

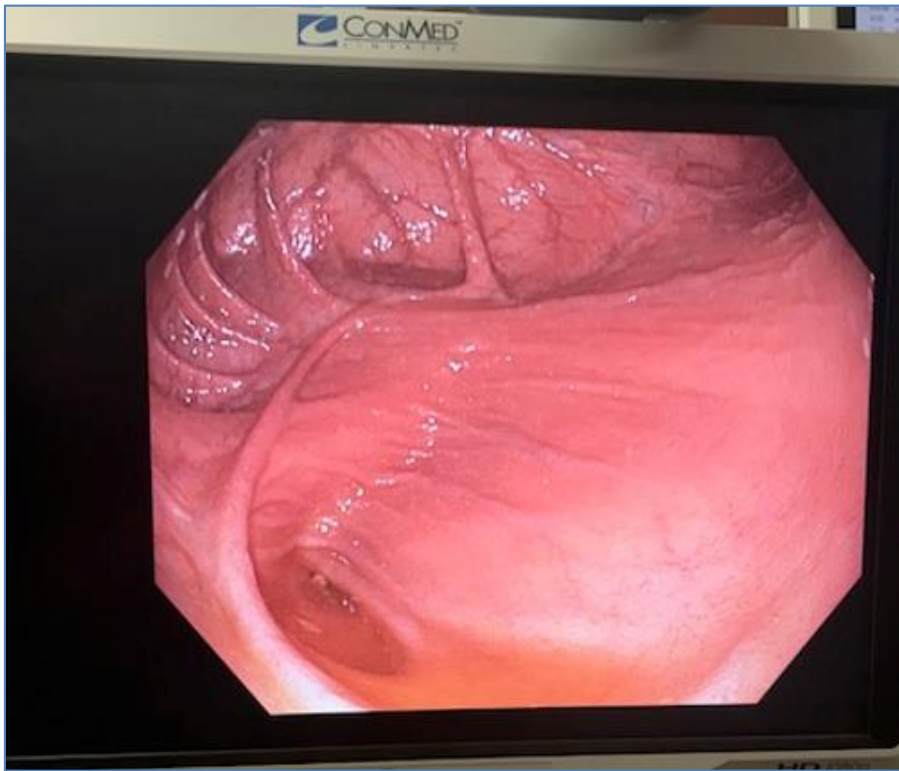

- Now go to the patient in ProVation.
- Select **GI** in ProVation and then **Procedure Documentation**.

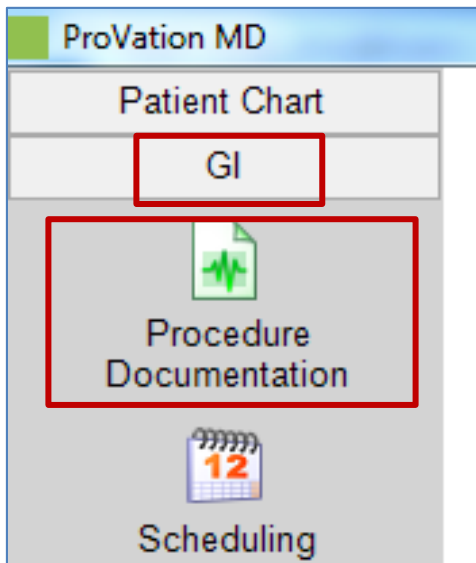


8. Select your Patient and press the capture button.

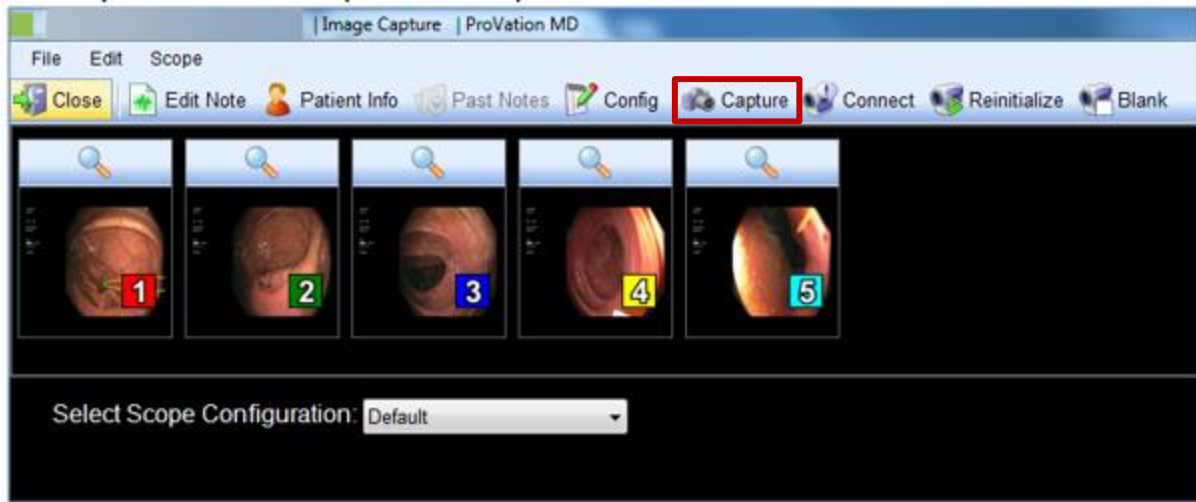


9. Once the above screen is up, **press P on the black keyboard** or select **CAPTURE** with the mouse.

10. After you have captured all the needed images, select **Close** in ProVation and go to your next patient.

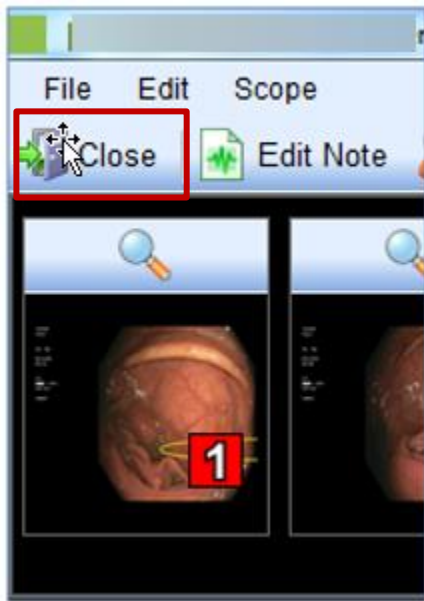


11. Now go back to the white keyboard and press **ESCAPE** all the way back to the main screen.


12. On the Olympus keyboard, hit the **Menu** button from the right side of keyboard.

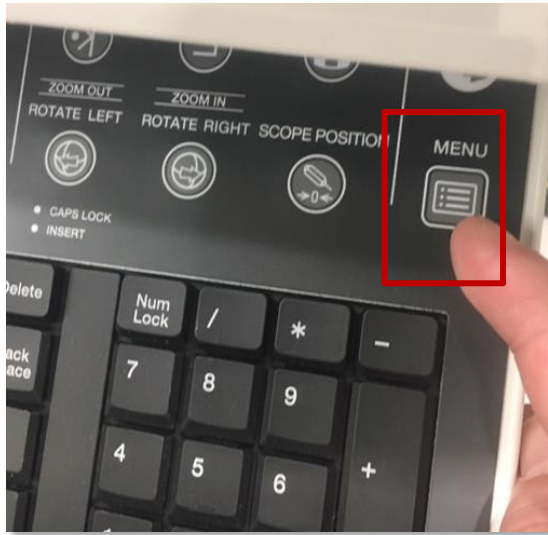


13. Use the arrow buttons to select **View Saved Images**. Hit enter.

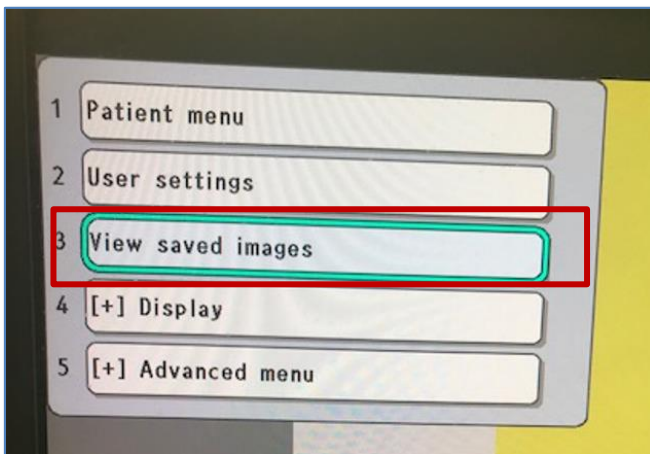

14. Use the arrow keys to navigate to **Manage Images**.

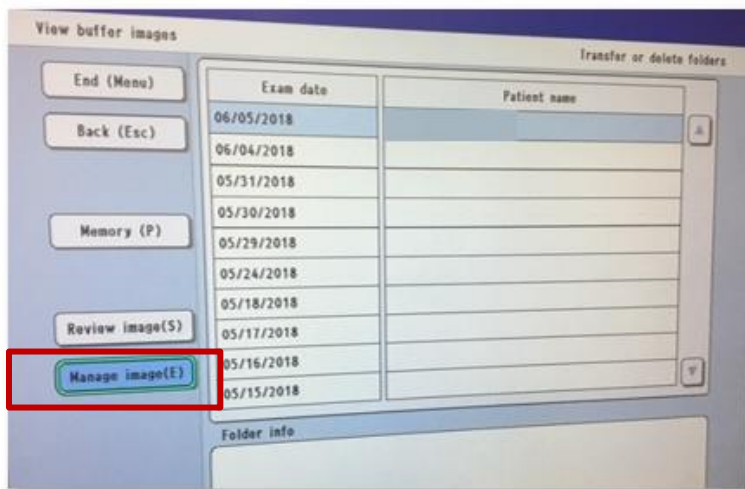

15. Use the arrow keys to navigate to **Delete** then over to **ALL**. Hit **R** on the keyboard.

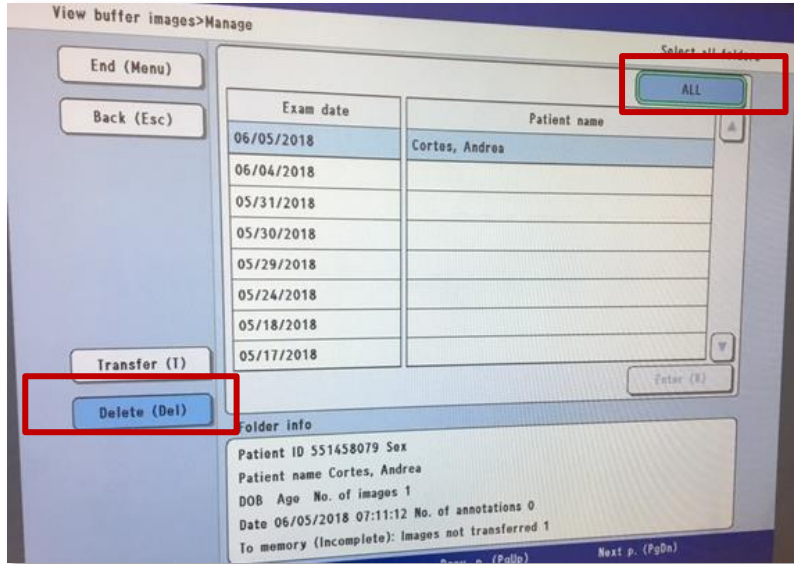

16. Select **Yes**. Hit Enter.